



# MedVizi™ App User Guide

## Download New Studies

Not for Clinical Diagnosis  
Copyright 2018 MedVizi, LLC - All Rights Reserved  
Version 0.4.2

**1** Type Study ID (case sensitive)

**2** Tap to List Downloads

**3** Tap 1+ Study View

**4** Tap to Download Selected Study Views

Tap Top of Screen to Hide Keyboard

Downloads Selects Available Studies

Create, Hide or Show Markup

**5** Tap-drag to Peel/Rotate Depending on the View

Pinch to Zoom

2-Finger Tap-drag to Pan

Viewer Interacts With Studies

### Filter Views By Study ID

Select Markups or Views

**6** Tap to Delete Existing Studies

Tap another View to Open in the Viewer

Studies Selects Downloaded Studies